

8*192 Thermal Compression Gloves

Replaces 8*199 gloves. (Sized Support)

NEW simplified design for comfortable fit.

- Front Grip where it is needed.
 - Streamlined smooth back of hand to reduce friction.
 - Relief from arthritic pain, cold and sensitive hands
 - Textured and fingerless gloves for sensitivity
 - Breathable and moisture wicking
 - Clinically validated
-
- Sizes are the same: XS, S, M, L, XL & XXL
 - Product pack stays the same: size 2
 - Product Dimensions: 125(H)x95(W)x40(D) weight 102, 104, 106, 108 & 110, 112gm

Materials:

- Nylon, Polyester, Spandex Rubber Foam



Actual Packaging



8*192 Thermal Compression Gloves

INSTRUCTIONS FOR USE

- Shaped for comfort, the Thermal Compression Gloves provide warmth and even compression for the temporary relief of arthritic pain.
- Designed for everyday use, the textured palms and fingertip cut outs provide additional grip and a better sense of touch so you can get on with your day-to-day without restriction.

WARNINGS

- Do not wear within the first 72 hours following an acute injury. This product contains a synthetic fibre, which may cause allergic reactions. Should an irritation or rash develop, discontinue use and seek medical advice. If pain persists, discontinue use and consult your medical professional.

CARE INSTRUCTIONS

- Wash before wear. It is recommended to place the product in a delicates washing bag. Machine wash with cold water. Drip dry. Do not wring, tumble dry, bleach or dry in direct sunlight.

WARRANTY DESCRIPTION

- Covered by a 12 month guarantee against faulty workmanship or materials

MATERIAL

- Outer lining: Nylon Mid layer: Rubber foam Inner lining: Polyester (Does not contain latex) - Made in Thailand

RELATED INJURIES

- Pain, stiffness or inflammation of the finger joints, Weakness in joints, Visible deformities in the fingers

INSTRUCTIONS

- Position the product on the right and/or left hand so that the fastener strap is wrapped around the lower forearm. Secure the strap.

OCCASION TO BE WORN

- Everyday, 72 Hours Post Injury



Actual Packaging

SIZING GUIDE

Measure circumference around knuckles

	SIZE	INCHES	CM
—	XS	6-6 3/4"	15-17
—	S	7-7 3/4"	18-20
—	M	8-9"	21-23
—	L	9 1/4"-10 1/2"	24-26
—	XL	10 3/4"-11 1/2"	27-29
—	XXL	11 3/4"+	30+

